NAPA-SOLANO-YOLO-MARIN COUNTY PUBLIC HEALTH LABORATORY 2201 COURAGE DRIVE, MS 9-200 FAIRFIELD, CA 94533 (707) 784-4410 FAX (707) 423-1979



Test	Measles (Rubeola) PCR
Test Description	Real-time reverse-transcriptase polymerase chain reaction (RT-PCR) for the detection of a target nucleocapsid gene sequence in the Measles virus genome. Note: Pre-approval by the Communicable Disease Department is necessary for testing.
Acceptable specimens	 Nasopharyngeal or oropharyngeal samples collected with a sterile synthetic swab (with an aluminum or plastic shaft) and placed in 2-3 mL of viral transport medium Urine at a volume of 10 to 50 mL in a sterile, leak-proof container
	Note: Dry swabs, swabs with calcium alginate tips, or swabs with wooden shafts are unacceptable for collecting specimens.
Timing of specimen collection	Specimens should be collected within 2 weeks of symptom onset.
Specimen storage	Specimens should be kept refrigerated at 2–8°C for up to 72 hours after collection, and frozen thereafter (repeat freezing and thawing is not acceptable).
Specimen shipping	Specimens should be shipped refrigerated (2–8°C) and received by the laboratory within 72 hours of collection.
CPT code	87798
Test fee	Refer to the posted fee schedule
Result availability	Results are typically faxed to the submitter within 1–2
(turnaround time)	business days after specimen receipt.